



REV.2024-06-04

INSTRUCTION FOR USE

Product Name:Ayre Spatula

Type:Gynaecological Wooden Cell-Collector F/S

Intend purpose: This product is suitable for diseases that require gynaecological examination.

Indications:Cell collector for cervical sampling.

Contraindications: If used on pregnant patients sample with care.

Caution: A. When the product is used, you should strictly implement the requirements of hygiene standards for gynaecological practice and related regulations, and be used by trained doctors.



B. The appropriate type of the Ayre Spatula should be selected according to the patient's body type. If you find any possible quality doubt during use, please stop using it immediately and notify our company/your dealer quickly.

C. This product is designed for single use only. The product is valid for 5 years and should not be used after expiring date.

D. If you notice burrs, cracks or splinters on the product, do not use the product.

E. This product shall be disposed of according to the requirements of waste disposal in the local medical regulations after using.

Patient group:It is suitable for people who require gynaecological examination.

Intended users:It can be used by trained doctors.

Clinical benefits: The product plays a supporting role in the detection of diseases.

Any residual risk and any undesirable side-effects: None

install information: None

Performance characteristics: Single use only.

Method:

a. Use the dilator to open the vagina at an appropriate angle.

b. Determine the area to be inspected or sampled.

c. Insert the Ayre Spatula into the sampling site and take a rotating method to sample.

Storage: The product should stored by room temperature.

Basic UDI-DI:6933298228



Shanghai International Holding Cord. Gmbh (Europe)Eiffestraße,80, 20537,
Hamburg, Germany Tel:+49-(0)40-2513175,Fax:+49-(0)40-255726



Jiangsu Suyun Medical Materials Co., Ltd.

No.18 Jinqiao Road., Dapu Industrial part, Lianyungang, Jiangsu Province, People's Republic of China.
Tel:0086-0518-85462073 Fax:0086-0518-85466033 Http://www.suyunmedical.com